

Otto Bock[®]

QUALITY FOR LIFE




Kiwi Pediatric Wheelchair

5 Point Harness Installation Instruction Guide

Step 1

Undo snaps on each side of the seat upholstery

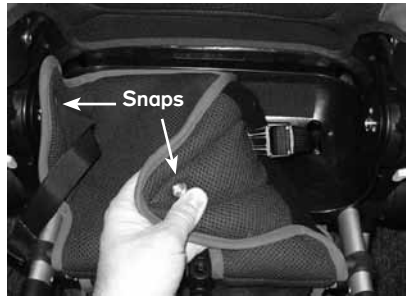


Step 2

Pull back the seat upholstery and remove the lap belt with buckles.




Step 3

Take new lap belt and weave it through the seat pan with the hook (velcro®) material facing upward in the middle slot.

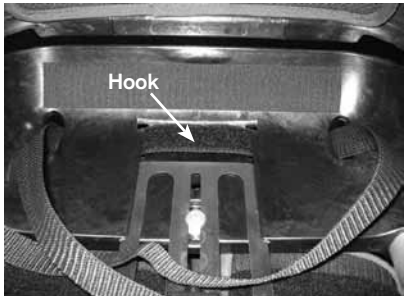


Step 4

From the bottom of the seat upholstery, put the loop end of the groin anchor strap through the center cut out and fasten the hook material of the anchor strap to the loop material of the seat upholstery.

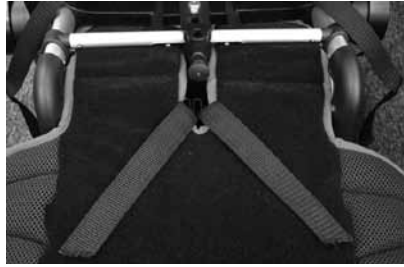

Step 5

Secure the square, hook material patch over the cut out

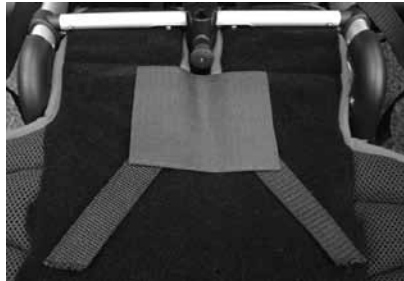


Step 6

Feed the lap belt ends through the cut outs on each side of the seat upholstery and snap the seat upholstery into place

Step 7

Attach the clip end of the groin strap to the groin anchor and snap the clip end closed

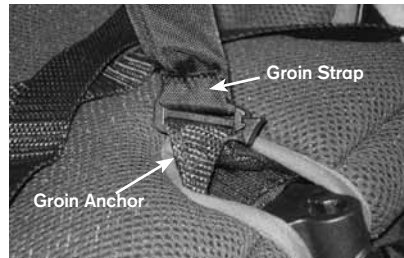


Step 8

Feed the lap belt end through the buckle as indicated on right.



Step 9

Repeat on other side

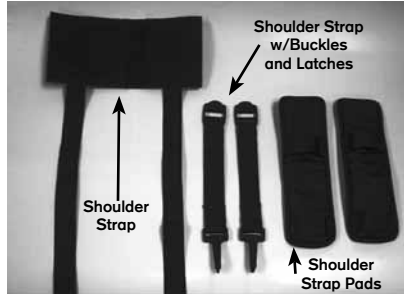

Step 10

Working from the back side of the Kiwi, un-Zip the upholstery




Step 11

Slide the shoulder straps through the desired slots of the back rest and attach the hook and loop material

Step 12

Zip upholstery closed

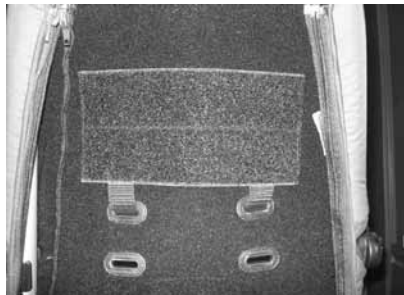

Step 13

Slide the shoulder straps with buckles and latches through the loops on the shoulder strap pads



Step 14

Feed the shoulder straps through the buckles of the straps w/buckles and latches



Step 15

Attach latch ends of shoulder straps to the buckles of the groin strap

Step 16

Place child in seat and adjust all straps to hold the child securely in the seat of Kiwi



Manufacturer:

Otto Bock Mobility Solutions GmbH

Lindenstraße 13 · 07426 Königsee/Germany

Phone +49 69 9999 9393 · Fax +49 69 9999 9392

ccc@ottobock.com · www.ottobock.com